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# Domotherm<sup>®</sup> EASY Digital Medical Thermometer

## Instructions for use

CE 0123

REF 0816  
PZN 10407062

### INTENDED USE

This digital thermometer is intended to measure the human body temperature in regular mode orally (in the mouth), rectally (in the anus) or axillary (under the arm). It is reusable for clinical or home use on people of all ages.

### SAFETY INSTRUCTIONS

- Measuring your temperature for self-diagnosis is dangerous! Please consult a doctor for an interpretation of the measurement readings. Self-diagnosis can aggravate an existing medical condition.
- Please read through these instructions carefully before using the thermometer.
- This device is not suitable for temperature measurement in the ear.
- For rectal measurement, do not insert the measuring tip with force. Discontinue the measurement if you feel any pain. Risk of injury!
- Do not use the thermometer for oral measurements if it has previously been used for rectal measurements.
- Do not conduct oral measurements on children under the age of 2.
- Do not bite the thermometer. This can cause damage or injury!
- Do not drop the thermometer. It is not resistant to shock or impact.
- Where protective coverings are used, the temperature measurement can deviate by around 0.1°C.
- The thermometer contains small parts (battery etc.) which children could swallow. Do not let children use the device unsupervised.
- Protect the device from high temperatures and direct sunlight.
- Do not expose the battery to extreme heat. Risk of explosion!
- Do not open the casing (except to change the battery).
- Using the unit in the immediate vicinity of mobile phones, microwave appliances or other devices with strong electromagnetic fields may result in impaired functioning. Maintain a minimum distance of 3 m from such devices when using this unit.
- Use outside the intended operating conditions or storage outside the intended storage and transport conditions can result in inaccurate measurements.
- Store the thermometer in the transparent box when not in use.
- Remove the battery when the device is not being used for a long period.

### MEASUREMENT TYPES

#### In the rectum (rectal measurement)

Apply some Vaseline to the measuring tip to aid insertion. The measuring tip is carefully inserted 1 cm into the anus.

#### In the mouth (oral measurement)

Insert the tip into one of the two heat pockets under the tongue, on the left- or right-hand side of the root of the tongue. Close the mouth and breathe calmly through the nose.

#### Under the arm (axillary measurement)

The skin must be completely dry. Position the measuring tip under your arm in such a way that the measuring tip is in good contact with your skin. Press your arm closely to your body. From a medical perspective, the axillary measurement method produces inaccurate measurement readings and should not be used if you wish to take precise measurements.

### MEASURING THE TEMPERATURE

To turn on the thermometer, press the on/off button. A short beep indicates that the thermometer is on. At the same time, an optical full segment check of the display will be carried out. Then the last temperature measured will be displayed for approx. 2 seconds, along with a small M (for memory). Then an internal test value appears and the thermometer switches to measuring mode.

Place the thermometer at the desired measurement point. During measurement the current temperature is shown on the display with a flashing °C symbol. A beep sound signals the end of the measurement. It is imperative you adhere to the minimum measurement time denoted by this signal tone. Please note that the thermometer will continue to measure after the signal tone.

For temperatures up to 37.8 °C, the normal sound will be made (10 long, equal beeps). For temperatures over 37.8 °C, the fever alarm will sound (30 short beeps in groups of 3). Read the temperature from the display. The thermometer will switch itself off automatically after approx. 10 minutes. You can also turn it off earlier by pressing the on/off button.

### CLASSIFICATION OF THE TEMPERATURES MEASURED

Body temperature measured orally is on average between 35.7 and 37.3 °C. For temperature classification, please refer to the following approximate values:

37.3 °C to 37.7 °C: elevated temperature

37.8 °C to 38.9 °C: moderate fever

39.0 °C and above: high fever

Temperatures measured rectally are generally 0.5 °C higher than those measured orally, while temperatures measured under the arm are 0.5 °C lower.

**Warning: Always consult a doctor if the reading shows a high fever.**

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7 0816 207 AA 2017-07

© 2014 UEBE Medical GmbH, Not to be re-produced, either in whole or in part, subject to technical modifications


UEBE Medical GmbH  
Zum Ottersberg 9  
97877 Wertheim, Germany

UEBE  
Germany est.1890

### CHANGING THE BATTERY

When the display shows the  symbol, the battery needs to be changed.

- Remove the battery compartment cover from the thermometer.
- Carefully pull the battery holder about 1 cm out of the casing.
- Please use a non-metallic pointed object to remove the battery from its holder.
- Insert a new battery (LR41 or SR41, 1.5V) with the + sign towards the top.
- Push the battery holder back into the casing and replace the battery compartment cover. Take care not to damage or misalign the seal when doing this.

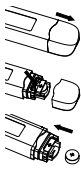### CLEANING

Clean the thermometer before and after every use, using a soft cloth and isopropyl alcohol diluted with water, or cold soapy water.

For cleaning, do not use petrol, thinner or any other strong solvent.

Do not immerse the device in alcohol for a prolonged period. Do not expose the thermometer to temperatures over 50 °C. Do not use ultrasonic cleaning.

### DISPOSAL

Batteries and technical appliances must not be disposed of with domestic waste, but should be handed in at the appropriate collection and disposal points.

### METROLOGICAL INSPECTION

Basically, a metrological inspection is recommended at intervals of 2 years. However, professional users in Germany have to comply with the aforementioned according to "Regulation for Operators of Medical Devices". This can be implemented either by UEBE Medical GmbH, an authority responsible for metrology, or authorised maintenance services. For this, please observe your national provisions.

### EXPLANATION OF SYMBOLS

CE This product complies with the Council Directive 93/42/EC from 5 September 2007 regarding medical devices and bears the mark CE 0123 (TÜV SÜD Product Service GmbH).

Degree of protection against electric shock: TYP BF

Please observe operating instructions

Storage and transportation conditions  
Ambient temperature -20 to 55 °C, rel. air humidity 15 to 95 %

Lot number

Manufacturer

DC direct current

### ERROR MESSAGES

Measured temperature is below 32.0 °C and thus outside the measurement range.

Measured temperature is above 42.9 °C and thus outside the measurement range.

Electronic error. If error persists, please contact UEBE customer service.

### TECHNICAL DETAILS

Type: Maximum thermometer, direct mode

Range: 32.0 °C to 42.9 °C

Accuracy: ± 0.1 °C between 35.5 °C and 42.0 °C  
± 0.2 °C in other temperature ranges

Operating conditions: Ambient temperature 15 to 35 °C, relative humidity 15 to 95 %

Storage and transport: Ambient temperature -20 to 55 °C, relative humidity 15 to 95 % conditions:

Battery: LR41 or SR41, 1.5 V

Battery life: Approx. 200 hours

IP rating: IP 27: protection from solid foreign bodies with diameters of 12.5mm and above, protection against temporary submersion

### WARRANTY

During the warranty period of 2 years from the date of purchase we reserve the right either to repair any such defect at our expense or to supply a perfect replacement unit. The cost of returning the unit to our factory shall be borne by the sender. UEBE shall refuse to accept return deliveries that have not been paid for by the sender.

Excluded from the warranty are parts subject to normal wear and tear as well as damage caused by non-compliance with the instructions for use, improper handling (e.g. unsuitable power sources, breakages, leaking batteries) and/or disassembly of the unit by the purchaser. Furthermore, no claims for damages against us are substantiated by the warranty.

Warranty claims can only be advanced in the warranty period and by presenting proof of purchase. In the event of a warranty claim, the unit must be sent to the following address together with the proof of purchase and a description of the complaint: Công ty TNHH YSMART, P1104-Nhà C5-Khu Đô Thị Mỹ Đình 1 - Q.Nam Từ Liêm - Hà Nội - Việt Nam. The statutory claims and rights of the buyer against the seller (claims for defect, manufacturer's liability, for example) are not restricted by this warranty.

**Please note: In the event of a warranty claim it is essential to attach the proof of purchase.**